

UNDERBODY WARMING

AIR-FREE / WATER-FREE WARMING MATTRESS OVERLAYS



CONDUCTIVE FABRIC PATIENT WARMING BLANKETS & MATTRESSES



PATIENT SAFETY

The conductive fabric in HotDog mattresses is designed to not interfere with the pressure reduction qualities of an OR mattress—a revolutionary advancement. Nearly all other underbody warming technologies are stiff, increasing the chance of patient bed sores via vascular occlusion and heat-accelerated tissue necrosis. HotDog does not increase vascular occlusion and is controlled at temperatures safe for under the patient.


COST

Most methods of underbody warming are outrageously expensive—often well over \$100 per patient. HotDog underbody warming can be as low as \$2 per patient. Cost considerations often force hospitals to warm only difficult cases, but one cannot always tell in advance which case will be difficult. With HotDog warming mattress overlays, all patients can be warmed at no additional cost.

EASE OF USE


With HotDog warming mattress overlays, there's no water—that means no leaking, no running out of water mid-case, no tech refilling the mattress during surgery. Unlike a large, rolling water-based machine, the small HotDog controller mounts on the IV pole. The system is easy to use and easy to clean. HotDog is the only patient warming system that warms from above (with blankets) and below simultaneously, making it the most complete solution.

SAFETY - PRESSURE

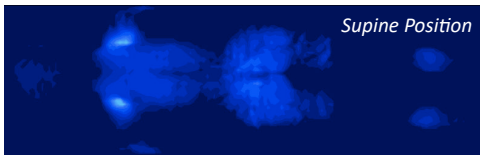

Supine Position

Pressure map: HotDog U102 Warming Mattress

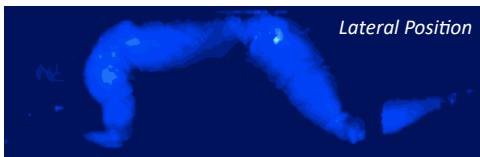


Lateral Position

Source: Augustine Biomedical + Design

The clinical consequence of high pressure is the possible development of a pressure ulcer in the soft tissue compressed beneath a bony prominence, especially in the presence of heat.

ThermAssure conductive fabric in HotDog is the only completely flexible fabric heater.

SAFETY - UNIFORM / SAFE HEAT

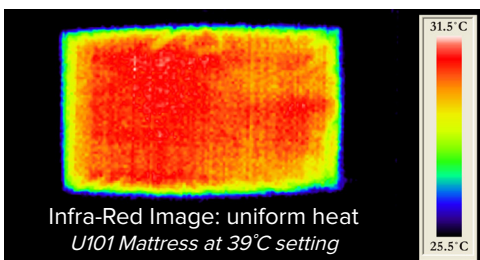

Infra-Red Image: uniform heat
U101 Mattress at 39°C setting

Safe Temperatures:
36-39°C (102.2°F)

With pressure, safe underbody warming should not exceed 40°C by ATM standards.

HotDog heats evenly across the full surface.

Additional Information

PNs and Dimensions	U101	U102	U300
	19.5 in x 32 in 49.5 cm x 81.3 cm	19.5 in x 50 in 49.5 cm x 127 cm	23 in x 39.25 in 58.4 cm x 99.7 cm
Underbody Warming Mattress			
Radiolucent	<i>Mostly</i> radiolucent. Only the buss bars along the long edge, the sensor and connecting wires are radio-opaque (green area).		
Construction	.75" pressure-relieving foam with flexible heater stretch over upper surface all within a sealed shell		
Versatility	Flexible overlay can be used inside of bean bags and peg boards. Also works with electrosurgical grounding pads and RF sponge counting pads.		
Antimicrobial Shell	Antimicrobial shield embedded in urethane shell		
Edge Seal	Crevice-free (RF-sealed)		
Energy Use	100-150 watts		
Ideal in:	Cardiac Surgery, Burn Units, Multi-procedure plastics, L&D (C-sections), Cath/EP Lab, Orthopedics		

www.hotdogwarming.com

Available on most major GPO contracts.

Rx Only

©2018 Augustine Temperature Management. All Rights Reserved.

6581 City West Parkway
Eden Prairie, MN 55344

952-746-1720 or 866-439-2767

AUGUSTINE
TEMPERATURE MANAGEMENT

PN M106C