



HotDog Training Warming Mattress + Return Electrode S.T.A.C.K.


Below are key reminders for safe use of capacitive return electrodes from HotDog. If you have additional questions, please contact your sales representative or contact us.

For safe use and product information, please scan the IFU QR code.



STACK

Set Up table stack with HotDog just below the draw sheet, drape, or inflatable transfer device.

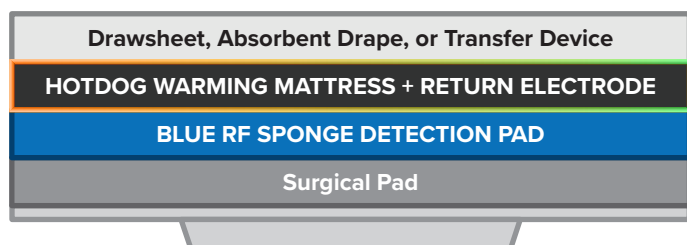

THIN BARRIER

Avoid thick layers between the patient and the HotDog mattress.




ALWAYS LOW

If typical, low ESU power settings no longer achieve the desired effect, first check the stack (good contact required).



CARDIAC

Protect, per protocol, patient/cardiac devices regardless of surgery location.

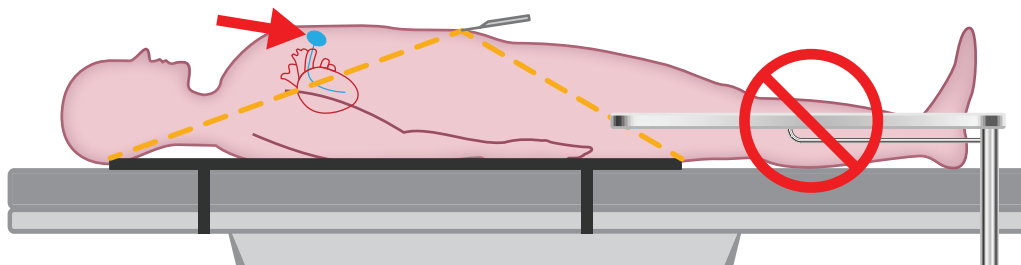

KEEP CLEAR

Keep the patient clear of contact with grounded metal.



Augustine
Surgical Inc.
Making Surgery Safer

www.hotdogwarming.com
©2024 Augustine Temperature Management.
All Rights Reserved.
Patents: hotdogwarming.com/patents
Rx Only
MT304EN Rev A 5/24

6581 City West Parkway
Eden Prairie, MN 55344
952-746-1720 or 888-439-2767
cs@aug surg.com