




HotDog Training Warming Mattress + Return Electrode


Always connect devices in this order.

1. Connect U5XX to A136/A137
- 2a. Connect A136 to Electrosurgical Generator or to A138 and then to the Generator.
or for dual setups:
- 2b. Connect A137 to Electrosurgical Generators or to A138s and then to the Generator.


Single Electrosurgical Generator

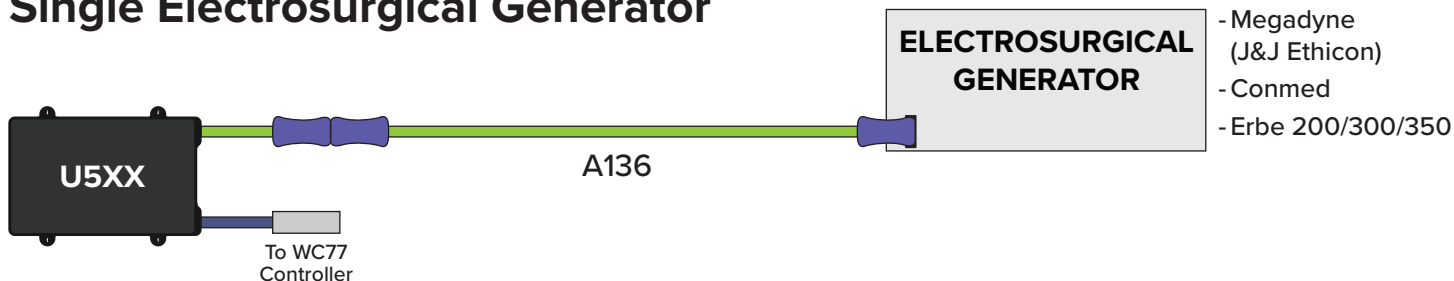


Single Electrosurgical Generator with REM

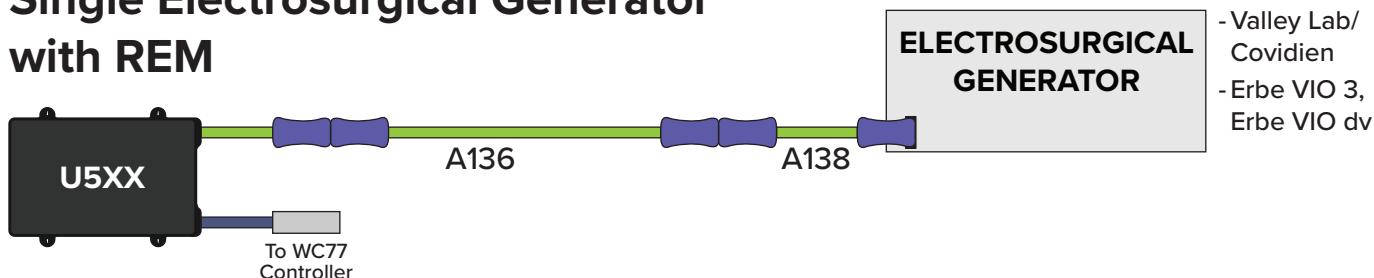

Dual Electrosurgical Generators

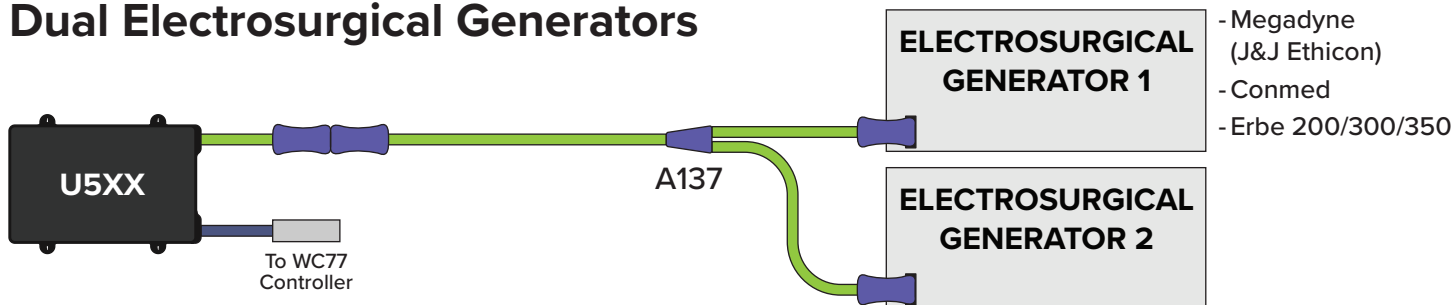

Dual Electrosurgical Generators with REM

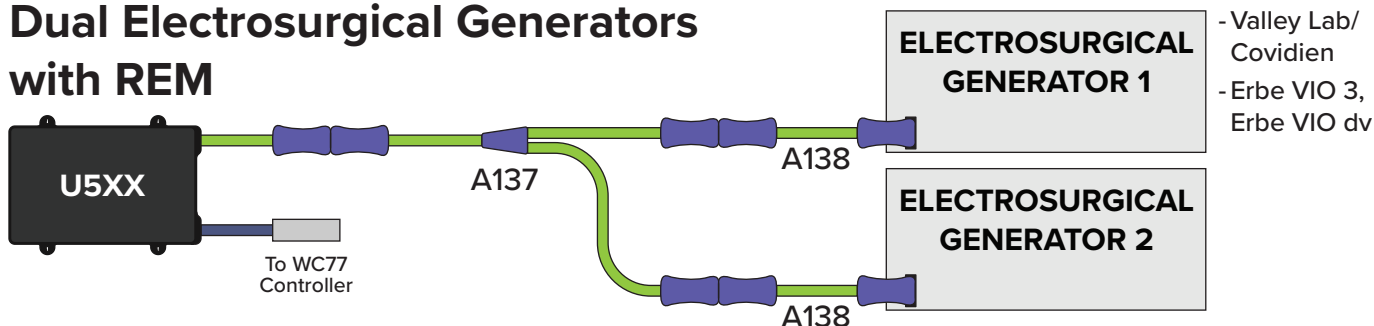



HotDog Product Details

Warming Mattress + Return Electrode

U501

82 cm (32")
0.9 kg (2.0 lb)



U502

127 cm (50")
1.5 kg (3.3 lb)



U530

89 cm (35")
1.0 kg (2.3 lb)



A136

Return Electrode Cable


13.1 ft (4.0 m)



A137

Return Electrode, Dual Generator "Y" Cable

11.5 ft, 4.9 ft (Y Lengths)
(3.5 m, 1.5 m)

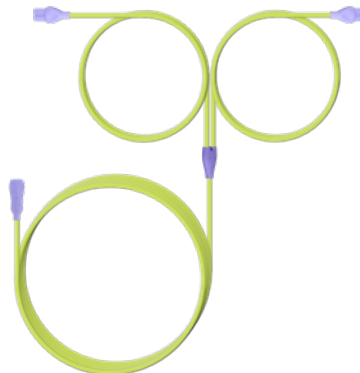

A138

REM Enabled Return Electrode Cable

16.5 in (42.0 cm)

