



# HotDog Positioning Guide Blankets and Mattresses

This positioning guide is intended to show the versatility of HotDog Patient Warming Blankets and Mattresses for **ALL** your surgeries and IR procedures and is by no means a comprehensive positioning guide.

For best results, warm above and below the patient simultaneously and cover as much surface area as possible while maintaining patient contact with sensor. Follow the **BEST** guidelines for optimal outcomes. Fold blankets as needed (Purple-on-Purple), do **NOT** fold black sides together. Position blankets with the power cable directed away from the surgical site. All Blankets and Mattresses are radiolucent.

